

Heart Saver AED7000


QUICKSTART USER GUIDE



Heart Saver AED7000

QUICKSTART USER GUIDE

Overview / Using Your AED

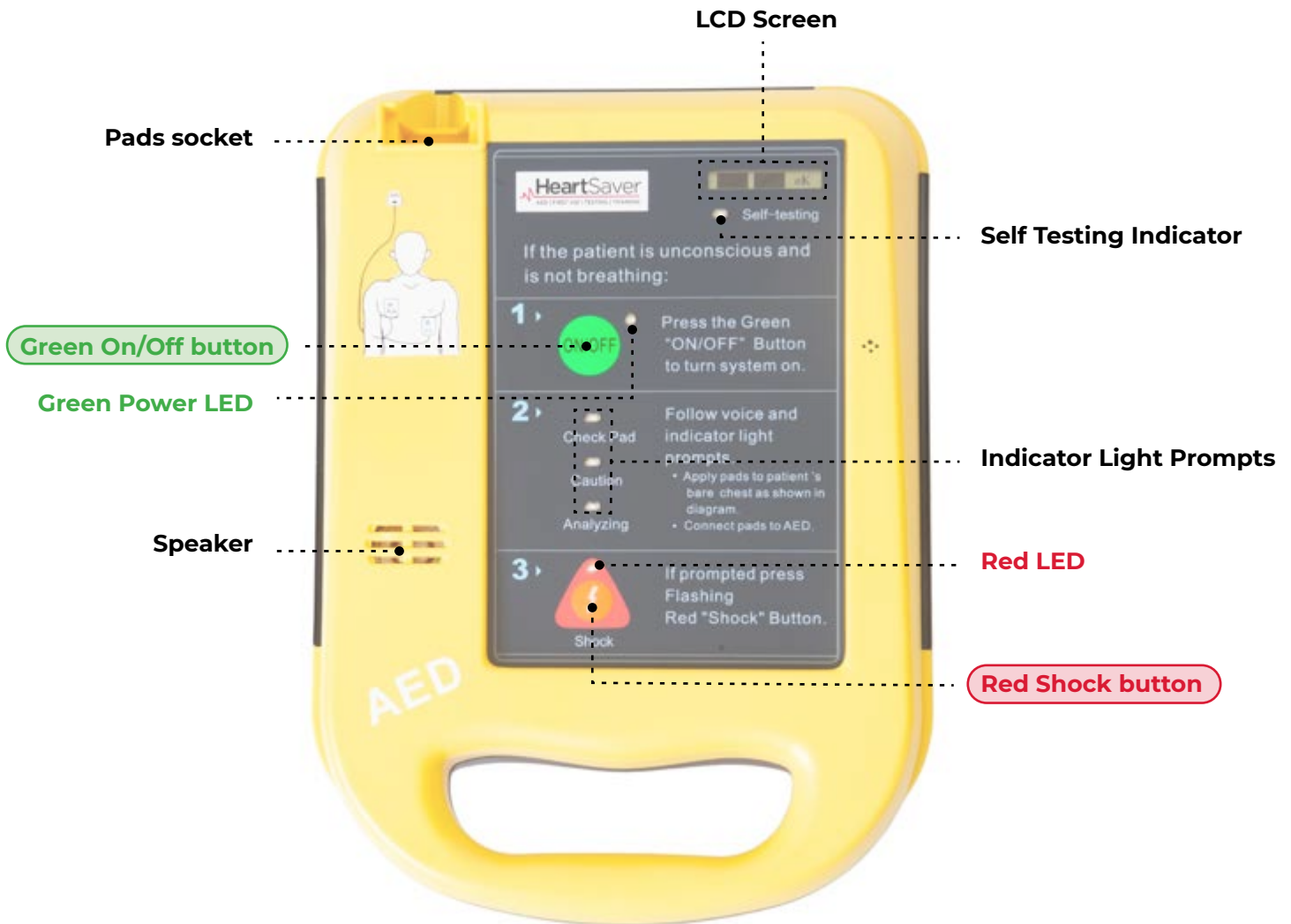

In case of emergency

1. Press **Green On/Off button** to turn the defibrillator on.
2. Follow the prompts.
3. If a shock is required you will be prompted to press the **Red Shock button**
4. Continue to follow the prompts.


Heart Saver AED7000

QUICKSTART USER GUIDE

LED / LCD Screen Indicators


LCD Screen displays self test results:

Low Battery

If the battery is low this icon will appear: 


[ORDER NEW BATTERY](#)

System Malfunction

If there is a fault this icon will appear: 

[CONTACT HEART SAVER](#)

System Normal

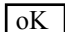If all is normal this icon will appear: 

NO ACTION REQUIRED

Green Power LED

- **LED is bright**
– Indicates the charge left in the battery is full
- **LED flashes slowly**
– Indicates the charge left in the battery is partially depleted
- **LED flashes fast**
– Indicates the charge left in the battery is low



Self Testing Indicator

On commencement of the daily/monthly self-test, this LED indicator will turn on; and automatically turn off on completion. Results will be displayed in the LCD.

Indicator Light Prompts

- **Check Pad LED Flashes**
– Defibrillator pads are not properly attached to the patient or are not properly connected to the AED7000
- **Caution LED Flashes**
– DO NOT TOUCH OR MOVE THE PATIENT
- **Analyzing LED Flashes**
– ECG leads are properly connected and the system is assessing the patient's heart rhythm

Red LED

flashes when defibrillator is fully charged.

Heart Saver AED7000

QUICKSTART USER GUIDE

Replacing the Battery

Battery replacement is recommended at 5 years. You can order a new battery for your AED7000 on our website by [clicking here](https://www.heart saver.co.nz/products/heart-saver-aed7000-replacement-battery) (or go to: <https://www.heart saver.co.nz/products/heart-saver-aed7000-replacement-battery>). Please follow the below instructions to replace the battery. You can also watch a short video [here](#).



To replace the battery follow the steps below:

1. Write the date of installation on the “New Battery Installed” sticker.
2. Open the battery compartment (located near the bottom of the backside of the AED7000) by removing the thumbscrews. Slip the plate up before pulling plate off.
3. Unplug the existing battery by gently pulling out the white 3-pin battery connector.
4. Turn the unit over and press the ON/OFF button. Wait until LED screen goes blank (you will still be able to see the battery/spanner icons). Leave unit like this for 60 seconds.
5. Connect the new battery by pushing the white 3-pin battery connector onto the contact pins.
6. Place the battery into the battery compartment.
7. Replace the battery cover by sliding the cover back so that the screw holes are aligned.
8. Replace the thumbscrews.



When you have installed the new battery, the AED unit will take a minute or two to boot back up. When it has completed its self-test, please check the LED screen on the front of the unit - you should be able to see “OK” and two black-shaded boxes (covering the spanner and battery icons).

PLEASE NOTE: Use only M&B Part No. CR123A-4x2 non-rechargeable lithium batteries. Use of any other battery can damage the AED7000.