

# Heart Saver AED7000


## QUICKSTART USER GUIDE



# Heart Saver AED7000

## QUICKSTART USER GUIDE

### Overview / Using Your AED

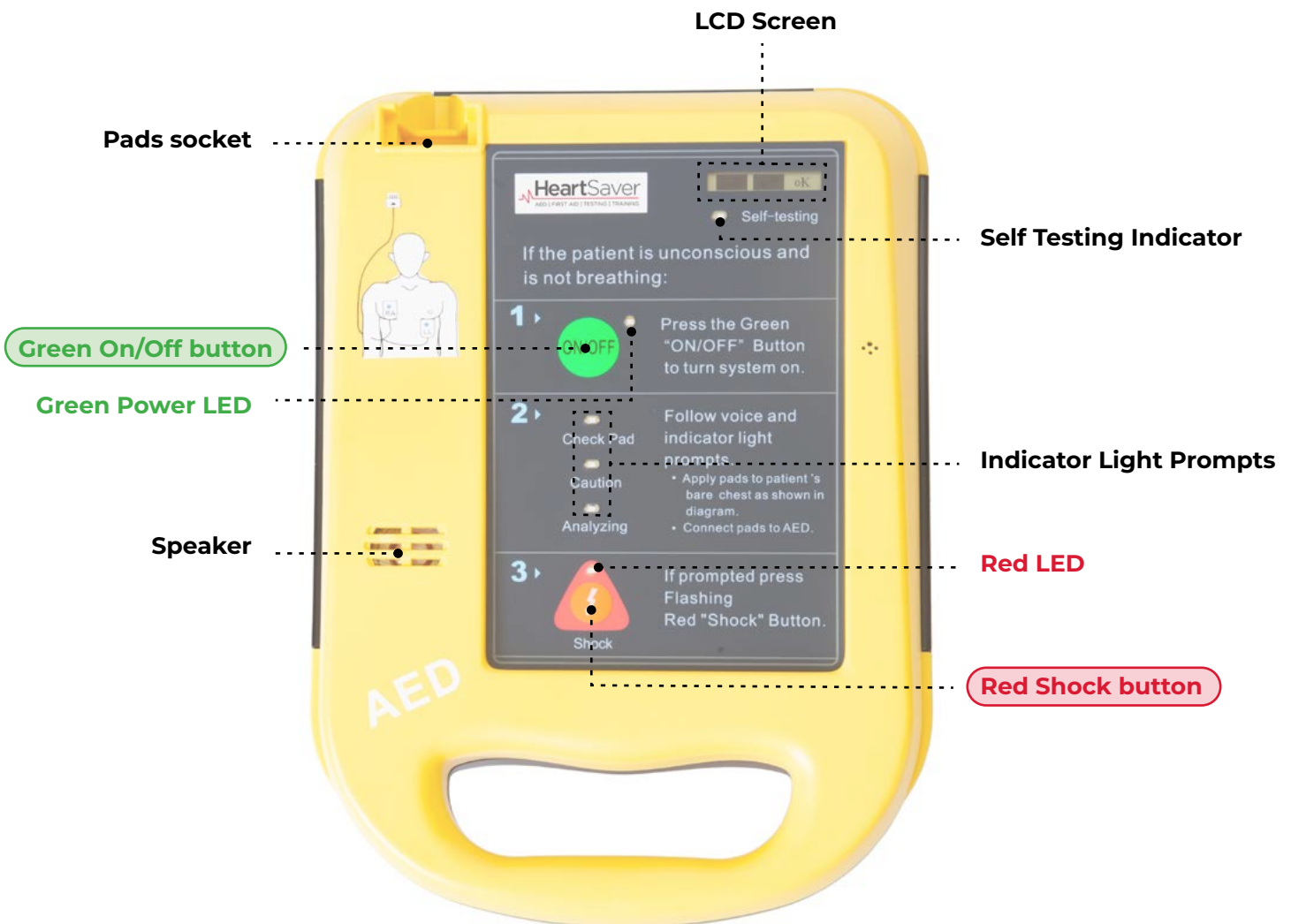

### In case of emergency




1. Press **Green On/Off button** to turn the defibrillator on.
2. Follow the prompts.
3. If a shock is required you will be prompted to press the **Red Shock button**
4. Continue to follow the prompts.

# Heart Saver AED7000

## QUICKSTART USER GUIDE


### LED / LCD Screen Indicators

**LCD Screen displays self test results:**

<b>Low Battery</b>	<b>System Malfunction</b>	<b>System Normal</b>
If the battery is low this icon will appear: 	If there is a fault this icon will appear: 	If all is normal this icon will appear: 
<a href="#">ORDER NEW BATTERY</a>	<a href="#">CONTACT HEART SAVER</a>	NO ACTION REQUIRED

#### Green Power LED

- **LED is bright**  
– Indicates the charge left in the battery is full
- **LED flashes slowly**  
– Indicates the charge left in the battery is partially depleted
- **LED flashes fast**  
– Indicates the charge left in the battery

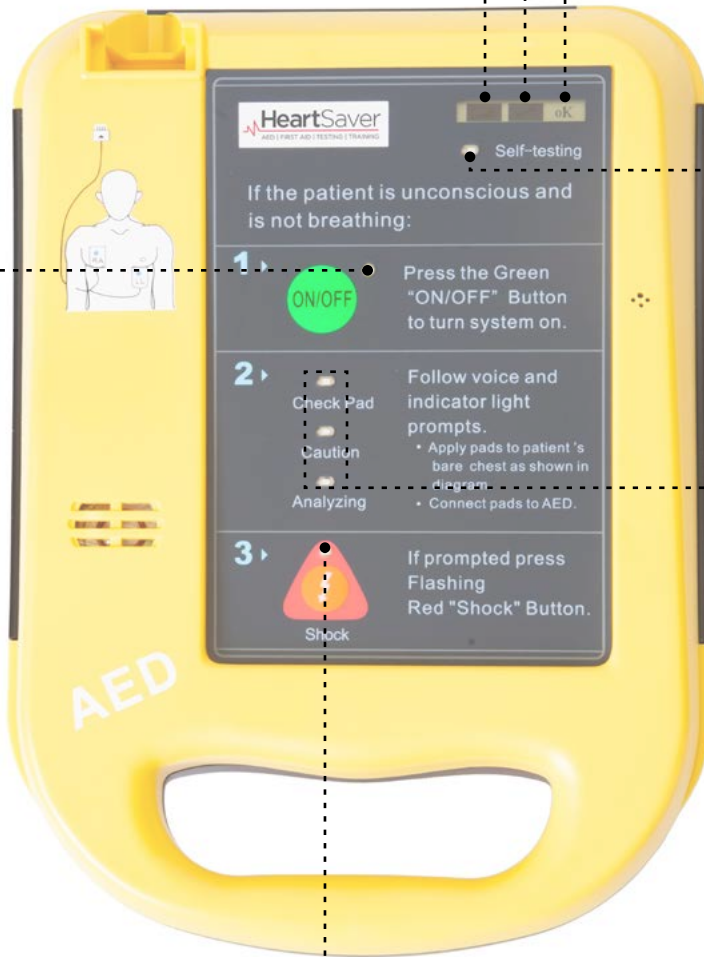

#### Self Testing Indicator

On commencement of the daily/monthly self-test, this LED indicator will turn on; and automatically turn off on completion. Results will be displayed in the LCD.

#### Indicator Light Prompts

- **Check Pad LED Flashes**  
– Defibrillator pads are not properly attached to the patient or are not properly connected to the AED7000
- **Caution LED Flashes**  
– DO NOT TOUCH OR MOVE THE PATIENT
- **Analyzing LED Flashes**  
– ECG leads are properly connected and the system is assessing the patient's heart rhythm

#### Red LED

flashes when defibrillator is fully charged.

# Heart Saver AED7000

## QUICKSTART USER GUIDE

### Replacing the Battery

Battery replacement is recommended at 5 years. You can order a new battery for your AED7000 on our website by [clicking here](https://www.heartsaver.co.nz/products/heart-saver-aed7000-replacement-battery) (or go to: <https://www.heartsaver.co.nz/products/heart-saver-aed7000-replacement-battery>)



**To replace the battery follow the steps below:**

1. Open the battery compartment (located near the bottom of the backside of the AED7000) by removing the thumbscrews



2. Slide the battery cover up and off.
3. Locate the 3 battery contact pins in the bottom-right corner of the battery compartment
4. Unplug the existing battery by gently pulling out the white battery connector



5. Connect the new battery by pushing the 3-pin battery connector onto the contact pins.
6. Place the battery into the battery compartment and replace the battery cover.
7. Slide the cover back so that the screw holes are aligned.
8. Replace the thumbscrews

**PLEASE NOTE:** Use only M&B Part No. CR123A-4x2 non-rechargeable lithium batteries. Use of any other battery can damage the AED7000.