




# Outdoor AED cabinet with keypad lock

## USER GUIDE

### Overview

- There are 10 number buttons and 1 letter button (Y) that can be used in your cabinet code.
- "C" is used to clear the key pad prior to loading your code.
- The cabinet code is factory set to C1.
- Codes should be reset by customers before installation.
- Codes can range from 1 to 11 digits.

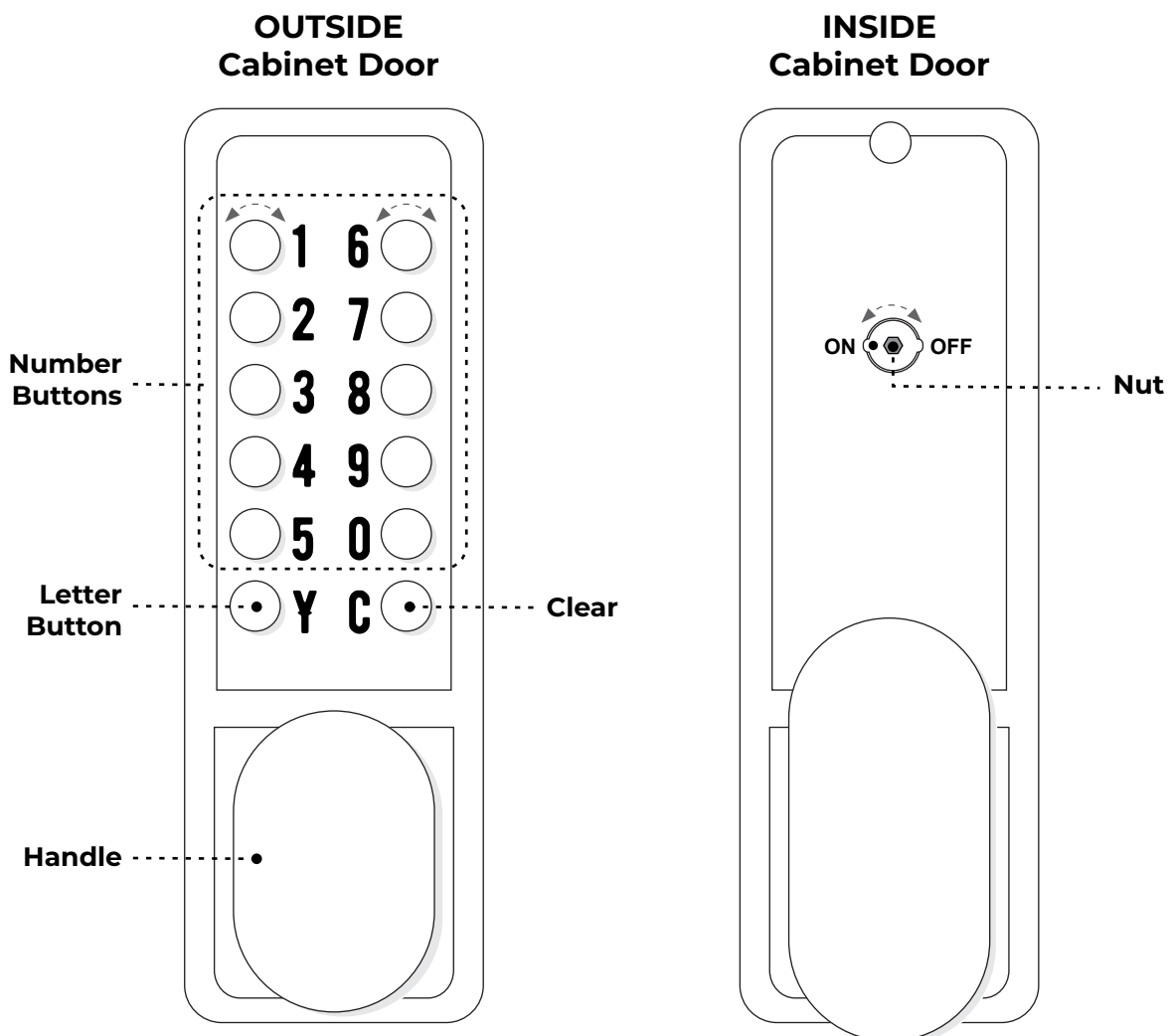

**Need help? Click here to watch a video**

or visit: <https://www.youtube.com/watch?v=BhNEQOWpRaw>

# Outdoor AED cabinet with keypad lock


## USER GUIDE

### To Set A New Code

1. Turn the handle and open the cabinet door. Use the Allen key provided to press and turn the inside nut from "OFF" to "ON"
2. The cabinet comes with a preset code of C1. To decode this preset, turn the number "1" in a clockwise direction 180°, ensuring the dots are in a horizontal position.
3. To set the new code, turn the desired numbers (in order) in an anticlockwise position by 180°, ensuring dots are in a horizontal position.

PLEASE NOTE: if code contains double numbers (eg. 223) only turn the "2" button once.

4. Return inside nut to "OFF" using the Allen key by pressing and turning to OFF position.
5. To open the cabinet push "C" and your code.

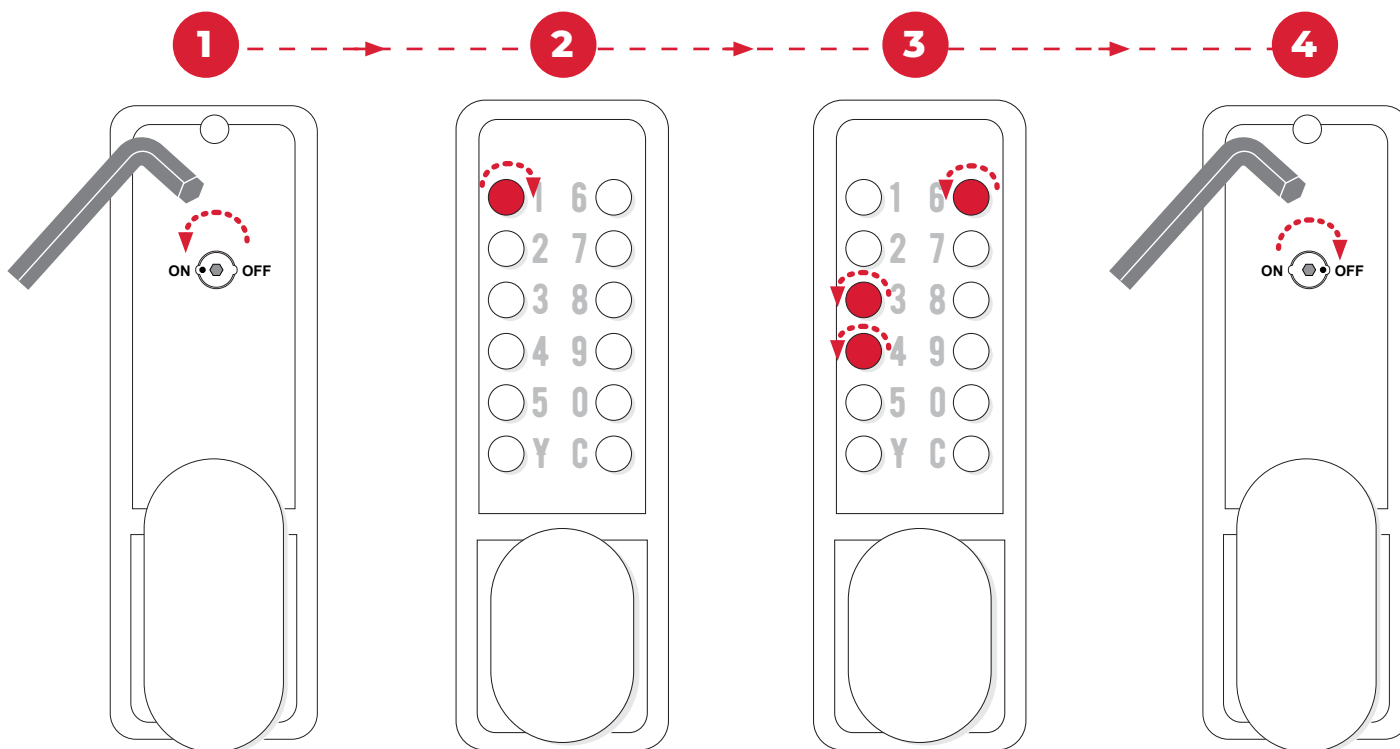

Please note - numbers above for example only. Choose code between 1 to 11 digits.

[Need help? Click here to watch a video](#)

or visit: <https://www.youtube.com/watch?v=BhNEQOWpRaw>

# Outdoor AED cabinet with keypad lock


## USER GUIDE

### To Change An Existing Code

1. Use the Allen key provided to press and turn the inside nut from "OFF" to "ON"
2. Turn the buttons of your current code in a clockwise direction 180°, ensuring the dots are in a horizontal position.
3. To set the new code, turn the desired numbers (in order) in an anticlockwise position by 180°, ensuring dots are in a horizontal position.

PLEASE NOTE: if code contains double numbers (eg. 223) only turn the "2" button once.

4. Return inside nut to "OFF" using the Allen key by pressing and turning to OFF position.
5. To open the cabinet push "C" and your code.

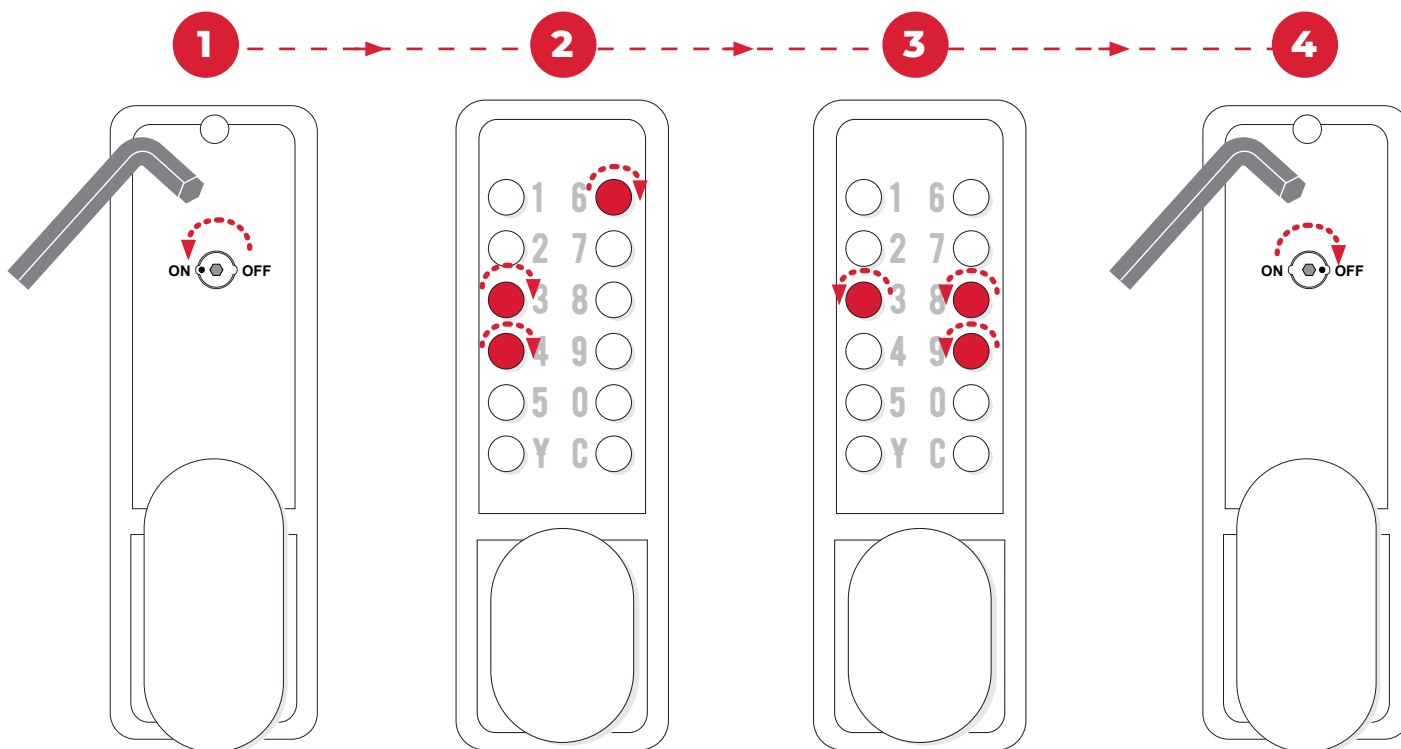

Please note - numbers above for example only. Use existing code numbers.

Please note - numbers above for example only. Choose code between 1 to 11 digits.

**Need help? Click here to watch a video**

or visit: <https://www.youtube.com/watch?v=BhNEQOWpRaw>